



**ALL-IN-ONE  
DISPOSABLE AV LINER**

**Including:**

**Liner, Filter and Collection Pouch**

**For Use with the  
MISSOURI MODEL AV**

**10 per Pack**

**Hamilton Research, Inc.**  
P.O. Box 2099  
South Hamilton, MA 01982  
(978) 468-4460, (800) 367-0266  
Fax: (978) 468-1359  
Equitainer@aol.com  
<http://www.equitainer.com>

### Instructions for Inserting the Missouri All-in-One Disposable Liner

1. Position the AV Liner so it matches the position of the AV (Fig. 1).
2. Gather the large end of the AV Liner and push through the small end of the AV. Reach in and pull AV Liner through the AV (Fig. 2). Pull until the collar and filter pouch of the AV Liner are firmly lodged against the smaller AV opening.
3. Make sure the liner is pulled smooth with no twisting. Fold the excess AV Liner over the top of the AV (Fig. 3) and tape securely (Fig. 4).
4. Tape the folded top of disposable liner over the top of the AV (Fig. 3).
5. Lubricate the disposable liner and finish preparing the AV as usual.

### After Collection

Fold a corner of the collection pouch, cut with scissors (Fig. 5) and gently pour out filtered ejaculate (Fig. 6).

### Notes

You may pour a known quantity of extender into the collection pouch of the Disposable AV Liner and collect directly into extender if advisable for the stallion. Remember to subtract the volume of extender added when measuring the ejaculate volume.

In cold weather, store the Disposable AV Liner in an incubator to avoid cold shock the sperm. Also, it is advised that an insulated cover be placed over the collection pouch.

