



**REGULAR  
DISPOSABLE AV LINER**

**Requires Addition of  
Filter and Collection Pouch**

**For Use with the  
MISSOURI MODEL AV**

**10 per Pack**

**Hamilton Research, Inc.**  
P.O. Box 2099  
South Hamilton, MA 01982  
(978) 468-4460, (800) 367-0266  
Fax: (978) 468-1359  
Equitainer@aol.com  
<http://www.equitainer.com>

## Instructions for Inserting the Colorado Regular Disposable Liner

1. Position the AV Liner so it matched the position of the AV. Fold over the small opening of the AV Liner and attach the wooden clothespin (Fig. 1).
2. Hold several inches of the AV liner in your hand, insert your hand into the AV and push the clothespin through the small opening in the AV (Fig. 2). From the outside of the AV, pull the clothespin and attached liner through the small opening.
3. Pull the liner from both ends to make sure it is smooth and straight.
4. Fold over the excess liner on both ends of the AV (Fig. 3).
5. Securely tape the AV Liner to the smaller end of the AV. Gently stretch the AV liner so it is tight and tape the other end in place (Fig. 4).
6. Place a filter over the small end of the AV and tape in place (Fig. 5).
7. Place a WhirlPak® Bag or baby bottle liner over the filter and secure with tape (Fig.6).
8. Lubricate the disposable liner and finish preparing the AV as usual.

### After Collection

Remove the collection bag from the end of the AV and transfer into a sterile specimen cup.

To remove the AV Liner after use, remove the tape from both ends. Pull the AV Liner through the small end of the AV to avoid getting semen on the interior of the AV.

### Notes

It is very important that the AV Liner is gently stretched tight, that it is straight and that it is securely taped in place. The AV Liner must not slide around in the AV.

In cold weather, store the Disposable AV Liner in an incubator to avoid cold shock the sperm. Also, it is advised that an insulated cover be placed over the collection pouch.

